


INTUITIVE
SURGICAL


SureForm™ 60

STAPLER USER MANUAL
INSTRUMENTS AND ACCESSORIES
USER MANUAL ADDENDUM

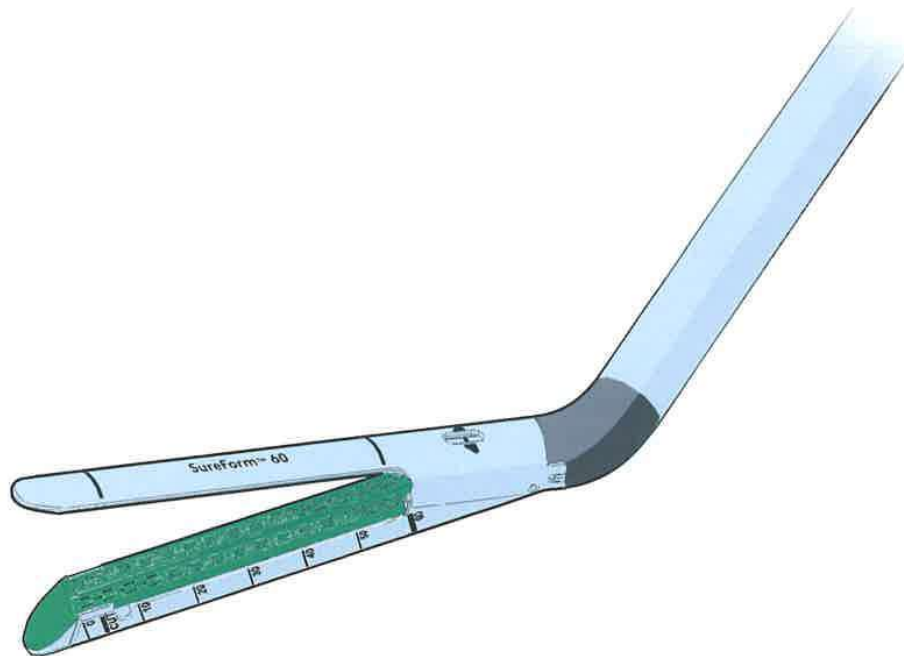

PN 553910-03 Rev. A 2018.03

 Intuitive Surgical, Inc.
1266 Kifer Road
Sunnyvale, CA 94086 USA
www.intuitivesurgical.com

 Intuitive Surgical, Sàrl
1, chemin des Mûriers,
1170 Aubonne Switzerland

 0543

DOCKET
ALARM

Find authenticated court documents without watermarks at docketalarm.com.

Copyright

© 2018 Intuitive Surgical, Inc. All rights reserved.

Trademarks

Intuitive, Intuitive Surgical, Beyond the Limits of the Human Hand, da Vinci, da Vinci S, da Vinci Si, da Vinci Xi, da Vinci X, ProGrasp, HotShears, EndoWrist, EndoPass, ClearField, CardioVac, Vacuum Source, SureForm, TilePro, and InSite are trademarks or registered trademarks of Intuitive Surgical, Inc. Other parties' trademarks are the property of their respective owners and should be treated as such.

Rx only

da Vinci Xi and *da Vinci X* SureForm 60 User Manual Addendum

Contents

1 Introduction	5
• 1.1 Contact Information	5
• 1.2 About This Manual	5
• 1.3 Indications for Use	6
• 1.4 Contraindications	6
• 1.5 Compatibility Information	6
• 1.6 General Warnings and Cautions	6
• 1.7 Device Description	9
• Articulation Overview	9
• 1.8 Stapler Components Defined	10
2 Preparation and Use	14
• 2.1 Open Packaging	14
• 2.2 Device Inspection	14
• 2.3 Spare Items and Emergency Equipment	14
• 2.4 Prepare for Intraoperative Use	14
• Install Reload	15
• 2.5 Intraoperative Use	16
• Precautions for Intraoperative Use	16
• Install onto Instrument Arm	17
• Calibration Sequence	18
• Clamp and Fire Overview	19
• Clamp and Fire Instructions	19
• SmartFire™	22
• Instrument Removal	24
• Remove and Replace SureForm 60 Reload	24
• Stapler Guided Tool Change	26
• Disposal	26
3 Troubleshooting	27
• 3.1 Stapler Initialization Failed	27
• 3.2 Missing or Used Reload	27
• Missing or Used Reload	27
• 3.3 SureForm 60 Reload Recognition	28
• SureForm 60 Reload Type Not Detected	28
• Dual Console	30
• 3.4 Wrong Reload Selected or Reload Shown Mismatch	30
• 3.5 Fire Unable to Complete	30
• 3.6 Mid-Procedure Restart	31
• 3.7 Blade Exposed	31
• 3.8 Force Unclamp	32

• 3.9 Manual Stapler Release Instructions	32
A Appendix A: MRI Safety Information	35
• RF Heating	35
• MR Artifact	35
B Appendix B: Symbols and Icons Reference Table	36
C Appendix C: Natural Rubber Latex	38
D Appendix D: Sterilization Methods	39

1 Introduction

This addendum to the *da Vinci*® System Instruments and Accessories User Manual provides details specific to the SureForm 60 and SureForm 60 reloads. This document is not a reference for surgical techniques. Refer to the appropriate *da Vinci* System User Manual, for full system information and warnings.

Failure to properly follow all instructions, including instructions supplied with the *da Vinci* System, may lead to injury and result in improper functioning of the device.

1.1 Contact Information

For Customer Service and Reporting of Complaints or Adverse Events

Use the following information for customer service, including ordering, reporting complaints or adverse events, and general information regarding *Intuitive Surgical* or our products and services.

In the U.S.

Intuitive Surgical, Inc.
1266 Kifer Road
Sunnyvale, CA 94086 USA
Toll free: 1.800.876.1310
Direct: 408.523.2100
Fax: 408.523.2377

In Europe

Intuitive Surgical
1, chemin des Mûriers,
1170 Aubonne, Switzerland
Toll free: +800.0821.2020
Direct: +41.21.821.2020
Fax: +41.21.821.2021

For Technical Support

If the system requires maintenance or service, please call our Technical Support line. In the US, call 1.800.876.1310, where phones are staffed 24 hours a day, seven days a week. In Europe, call +41.21.821.2020.

Manufacturer

	<p>Intuitive Surgical, Inc. 1266 Kifer Road Sunnyvale, CA 94086 USA www.intuitivesurgical.com</p>
---	---

1.2 About This Manual

This manual presents representative images of the stapler instrument. The representative images provide instruction on the use of the product. In some cases, the aesthetics or length of the device shown in the representative images may be different from the product in use.

Explore Litigation Insights

Docket Alarm provides insights to develop a more informed litigation strategy and the peace of mind of knowing you're on top of things.

Real-Time Litigation Alerts



Keep your litigation team up-to-date with **real-time alerts** and advanced team management tools built for the enterprise, all while greatly reducing PACER spend.

Our comprehensive service means we can handle Federal, State, and Administrative courts across the country.

Advanced Docket Research



With over 230 million records, Docket Alarm's cloud-native docket research platform finds what other services can't. Coverage includes Federal, State, plus PTAB, TTAB, ITC and NLRB decisions, all in one place.

Identify arguments that have been successful in the past with full text, pinpoint searching. Link to case law cited within any court document via Fastcase.

Analytics At Your Fingertips



Learn what happened the last time a particular judge, opposing counsel or company faced cases similar to yours.

Advanced out-of-the-box PTAB and TTAB analytics are always at your fingertips.

API

Docket Alarm offers a powerful API (application programming interface) to developers that want to integrate case filings into their apps.

LAW FIRMS

Build custom dashboards for your attorneys and clients with live data direct from the court.

Automate many repetitive legal tasks like conflict checks, document management, and marketing.

FINANCIAL INSTITUTIONS

Litigation and bankruptcy checks for companies and debtors.

E-DISCOVERY AND LEGAL VENDORS

Sync your system to PACER to automate legal marketing.